

*VetScan*  
**VS2**



## Important Notification!

### VetScan VS2 Software Upgrade

**Software Version 2.1.31**

# Important Software Upgrade Notification

---

Dear VetScan VS2 Customer:

**Enclosed is your VetScan VS2 software version 2.1.31.**

Included in this release are:

- Enhanced Calcium analyte calibration factors.
- Further expansion of intelligent Quality Control (iQC™)

Therefore, installing this latest version of software is required for optimal use of your VetScan VS2 Chemistry Analyzer.

***Note** - this software upgrade does not affect connectivity or stored information. However, if your analyzer is connected to a PMS or LIS, as a precaution, please notify your PMS/LIS provider of the software upgrade to ensure uninterrupted Data transfer.*

For simple instructions on upgrading your software, please refer to the attached “Software Update Instructions” or the “Maintenance and Service” section in your VS2 Operator’s Manual.

Enclosed with this letter:

1. Software Version 2.1.31 CD
2. Software Upgrade Instructions

If you need assistance with updating your software, please contact Abaxis Europe Technical Support at +49 6151 350 790 or email [techsupport@abaxis.de](mailto:techsupport@abaxis.de).


Thank you for using Abaxis to fulfill your in-office testing needs.

Sincerely,

Abaxis Europe GmbH

# Software Upgrade Instructions

---

1. In the Home screen, press the **Settings** icon.....
2. Press the **More Settings** icon.....
3. Press the **Software Update** icon.....
4. Place the CD in the drawer, press the drawer inward to close it
5. Then press **Continue**.....  
(Press the Information icon  for additional help)



The software update will automatically begin. When the update is complete, the analyzer will automatically shut down and restart.

**CAUTION: TO AVOID DAMAGE TO YOUR ANALYZER, do not remove the CD or turn the power off while the analyzer is performing the software upgrade. Allow the analyzer to complete the full upgrade cycle and restart.**

To verify the software has been successfully updated, see the Operator's Manual, Section 4.6, page 4-1. Software installation instructions can also be found in the VetScan VS2 Operator's manual; section 9.4, pages 9-3 to 9-4

Custom-fitting diagnostic solution in  
animal health care,  
ABAXIS VetScan® Systems



*VetScan*  
**VS2**



*VetScan*  
**HM5**<sup>TM</sup>



*VetScan*  
**i-STAT1**<sup>®</sup>

ABAXIS Europe GmbH  
Pekapark, T9  
Otto-Hesse-Straße 19  
D-64293 Darmstadt  
Germany  
[www.abaxis.de](http://www.abaxis.de)

+49-6151-350-79 0